

**BMOIST Instruction for Use**



Figure 1

**BMOIST**



Figure 2

Position of Heating Box on the clip.



Figure 3

BMOIST connected to the power bank.



Figure 4

Power Bank with the charger cable



Figure 5

BMOIST Power Bank Charger

Doc # 1026280824

**Intended Use**

The BMOIST powered eye warmer device is intended for providing localized heat in the eye area to relaxation. The device is not intended for therapeutic purpose.

**How it works**

BMOIST works solely heating the eyelids and is thus completely free from chemicals. The local heating is a physical mode of action based on a short (5 min.) application of heat (max. 45°C) to the eyelids, sufficient to trigger a response from the body to provide hydration of the eye.

**Who should NOT use BMOIST.**

- who past an eye surgery and the post-surgery symptoms not passed yet.
- who have, an inflammation of the eye such as conjunctivitis, keratitis and anterior uveitis or if you cannot close your eye.
- Immediately after application of ointments, creams, lotions, etc.

**Frequency of use**

- 2-3 times a day but not more than 5 times with minimum interval of 4 hours.

**BMOIST package contains (Fig.5):**

- BMOIST (fig1)
- Power Bank (Fig.4)
- Charger cable Fig.4)
- Instruction for User
- Power Bank Manual

**Before using BMOIST:**

- verify that that heating box and slider are not broken.
- charge the Power Bank (P.B). During charging the P.B, the incorporated red lamp is flashing. The P.B. is full charged when this red lamp stops flashing.

**Directions for use**

1. Without connecting to P.B, place the heating box on your glasses (Fig.2) by sliding the heating box onto the left or right eyeglass lens with red side directed to the eye.
2. Put the glasses on and verify that the heating box is below the eyebrows. If the red heating box is too high, push it down. After you placed the BMOIST in this manner on your glasses take them off.
3. Clean the eyelid from makeup.
4. Connect the USB cable to the Power Bank. Verify that the red panel is illuminated it indicating that BMOIST is activated. (Fig. 3).
5. **Close the treated eye** and put on your glasses with BMOIST attached.
6. The session of each eye lasts about 5 minutes. When session ends, the red panel illumination will turn OFF, indicating that BMOIST is inactive.
7. To reactivate BMOIST reconnect it to the Power Bank.
8. Repeat the same procedure for the other eye if necessary. It is recommended to wait 5 minutes for the second eye sessions.
9. Once session ends, take off glasses, remove BMOIST, disconnect Power Bank and place all back in the carton box.

**Cleaning information**

To clean the red surface, use a small cloth with soap and water. Don't use detergents such acids or alkalis or alcohol that can harm the eyelid. Do not use objects that could damage or scratch the red surface.

**Caution**

- It is forbidden to use BMOIST while charging the power bank!!
- Please report any serious incidents related to the device to both the manufacturer and the competent authority in your state.

**Note**

- BMOIST must be disconnected from Power Bank when not in use.
- Stop the session if you feel uncomfortable or if a change in BMOIST performance is observed!
- Use BMOIST to relax dry eyes only. Don't use the BMOIST in a humid environment such as bath, rain heavy fog.
- Do not try to repair the device on your own.
- Do not give, sell, rent, or allow the use of your device to or by another person.
- Do not modify this device.
- Protect the device from high humidity and contact with liquids.

**Specifications**

Power Supply – 5Vdc Power Bank is deliver  
 Power Bank must be used according to its manual.  
 Power consumption – max 1W  
 Heat source – IR LED  
 Max/Min operation ambient temperature - 35°C/0°C.  
 Max/Min Sstorage and Transportation ambient temperature - 60°C/-20°C.  
 Humidity operation max. - 75%.  
 Max. panel temperature - 45°C.  
 Continuous operating time – 5 min.  
 Size of heat box – 45x30x15mm  
 Cable length – 1m  
 Connector type USB C female  
 Weight without Power Bank – 32 gr.  
 Weight of packing – 218 gr.  
 Min glasses thickness - 2mm.  
 Max glasses thickness - 5.5mm.  
 Body Material – plastic MABS  
 Compliance with following Standards: IEC 60601-1; IEC 60601-1-11; IEC 60601-1-2; IEC 60601-2-57; IEC 62471; ISO 10993-1; UL94; IEC 62133-2.

**Storage instructions**

Store the BMOIST protected from direct sunlight and in a dry place. Do not use the Power Box beyond the expiry date stated on the label of P.B (

**Symbols on the BMOIST**

The following symbols appear on the BMOIST device:

	CE mark for Europe.		Temperature Limit
	electronic devices dispose		Fragile, handle with care
	Manufacturer		Country of manufacture Date of manufacture
	Medical Device		Unique Device Identifier
	Consult instruction Medical Device		Single User
	Keep dry		Lot Number
	Lithium Battery Shipping Guideline		Follow instructions for use
	Catalogue Number		Humidity Limitation
	Authorized representative		

**Waste Electrical and Electronic Equipment (WEEE) Policy**

The European Directive 2012/19/EC on WEEE requires proper disposal of electrical and electronic equipment when it reaches its end of life. The crossed-out wheeled bin symbol indicates that this product must not be disposed with other waste; it must be taken to an approved treatment facility or to a designated collection point for recycling, according to local legislation.

**BMOIST Warranty** The manufacturer provides 12 months warranty.

**For product and service inquiries, please contact:**

**Manufacturer:**

MHeye Ltd.  
 35 Givati st Rishon Lezion 7533605, ISR  
 Tel: +97239664246

E-mail: office@the-moisteye.com

**Date of revision of the Instruction for use: 25.09.2024**